

Update 26.0.0 qube**SERIES**

Release Notes Version 26.0.0

General

- Stability and performance optimizations
-

New interface

- **qubeD60-DX** — simulates the BeneHeart D60 & DX from Mindray

qube**D60-DX**




qubeC3/C3T

- Due to a manufacturer change in the defibrillation and AED modes, the CPR waveform field has been removed from the default view. This can be changed again in the layout editor in **qubeCLOUD** at your discretion.

Required steps:

1. Log in to **qubeCLOUD**
2. SIMCONFIG
3. Select monitor
4. DEFI -> DEFINE LAYOUT

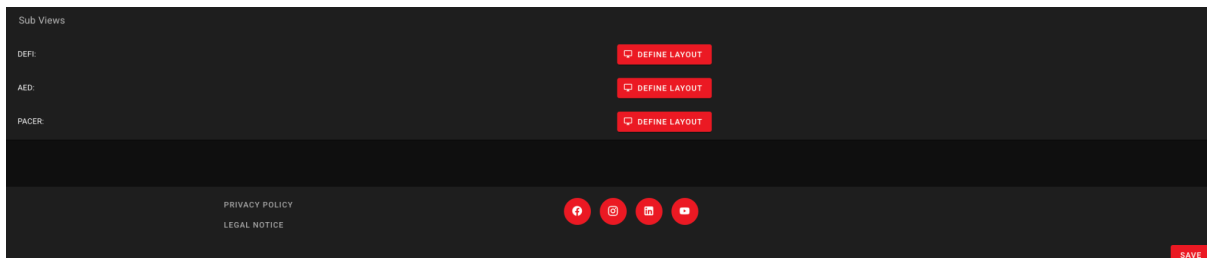

5. Change layout and SAVE
6. Also SAVE in SIMCONFIG
7. Connect **qubeCONTROL** to an internet-enabled Wi-Fi network and synchronize
8. Connect **qubeMONITOR** to **qubeCONTROL**

qubeCONNECT

- Our new WebSocket connection allows third-party providers to seamlessly integrate their simulation systems into our **qubeSERIES**. Vital signs can be displayed on monitors and ventilators. Interactions with these devices, such as defibrillation or ventilation, can be simulated on the connected devices.

AmbuMan interface

- The interface to AmbuMan has been fundamentally redesigned for better performance, taking customer feedback into account.

Important note

- When upgrading to version 26.0.0, please ensure that you perform the update on all iPads in use.
- The change also requires an update of the **qubeBOX** firmware to version 1.2.3. To do this, all sensors must be disconnected during the update. Instructions for the update can be found [here](#).

Contact
support@skillqube.com
+49 6222 – 386265
skillqube.com/support