

Update 26.1 qube**SERIES**

Release Notes Version 26.1

General

- Stability and performance optimizations
-

New interfaces

- **qubeEMV+** — simulates the EMV+ from ZOLL
- **qubeZVent** — simulates the Z Vent from ZOLL

qube**EMV+**




qube**ZVent**




qubeC3/qubeC3T

- **CPR Reference Range:** The reference range for the CPR curve can now be adjusted between the default value (5–6 cm) and a reduced depth (4–5 cm).

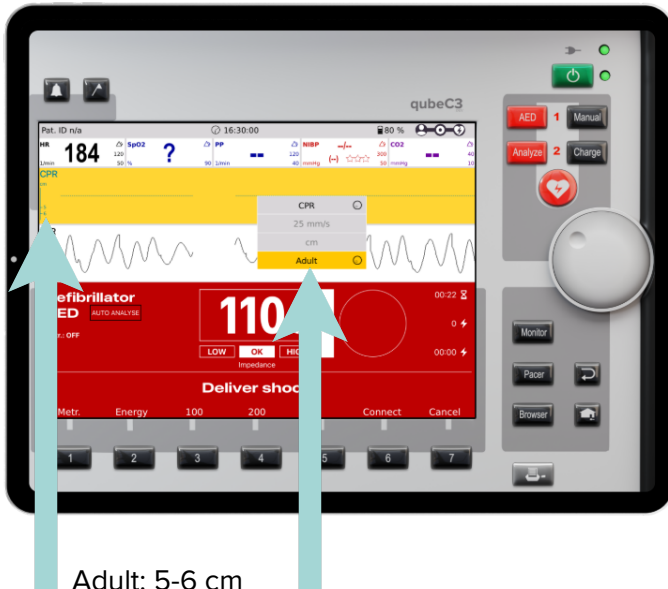

Adult: 5-6 cm

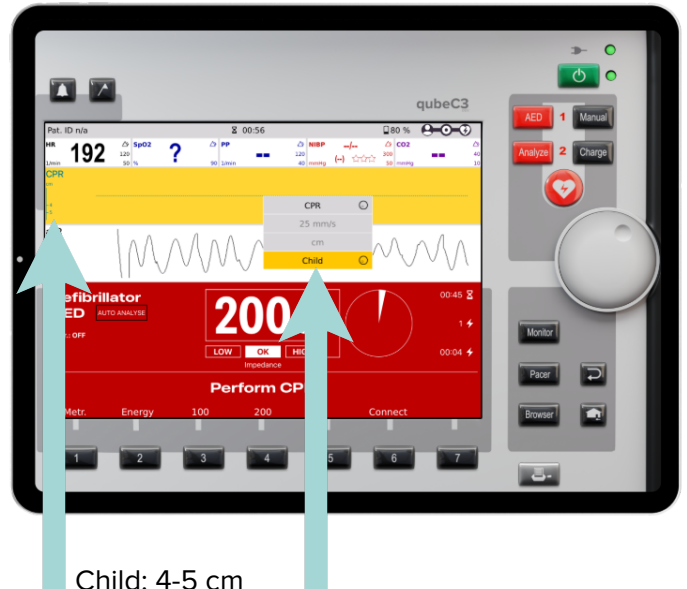


Child: 4-5 cm

- **Optimized NIBP quality indicator:** The display of the non-invasive blood pressure (NIBP) measurement quality via the star indicator has been improved.

qubeCONTROL

- **Note on hard paddle support:** If a monitor in AED mode does not support hard paddles, this is now immediately signaled to the instructor within **qubeCONTROL**.

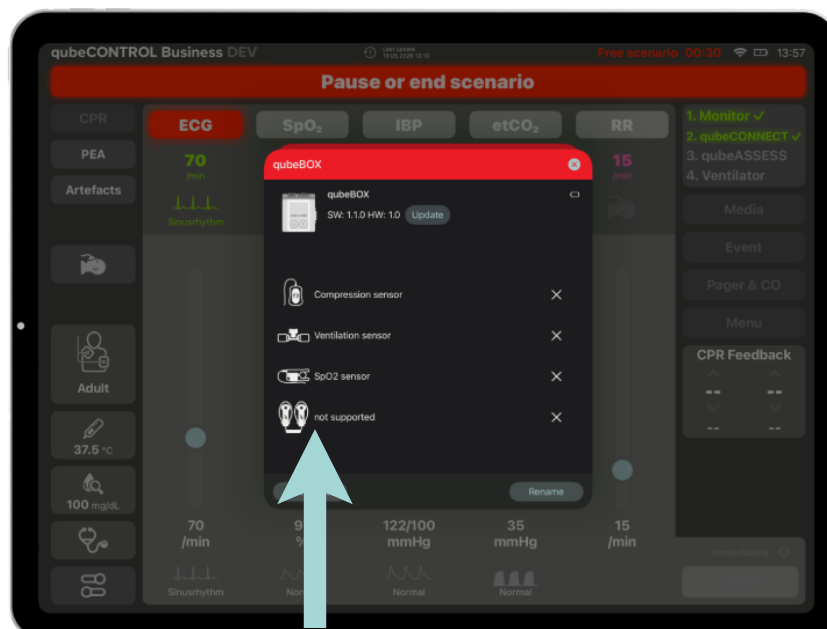

SKILLQUBE


- **Expanded patient details:** In predefined scenarios, the patient's name and age are now displayed directly in **qubeCONTROL**.



- **Flexible scenario sorting:** In the scenario overview, scenarios can now be sorted by name or modification date (both ascending or descending).

qubePMD / qubeR / qubeX

- **Detailed CPR feedback:** The CPR feedback display has been optimized, providing a more precise, detailed visualization of chest recoil (pressure relief).

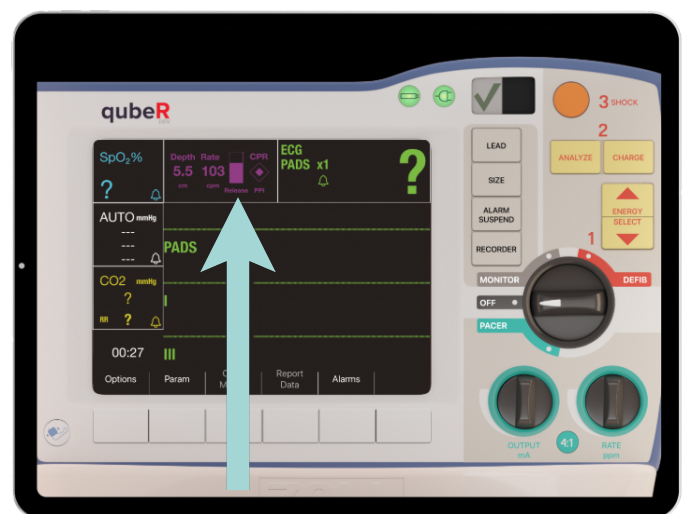

Important note

- When upgrading to version 26.1, please ensure you update all iPads in use.
- A **qubeBOX** firmware update to version 1.2.3 is still required, if not already done.
 - Upgrading from **qubeSERIES** version 6.1.0 or older: Please disconnect all sensors during the firmware update. Instructions for the update can be found [here](#).
 - Upgrading from **qubeSERIES** version 6.2.0 or newer: Sensors no longer need to be disconnected for future **qubeBOX** updates.

Contact

support@skillqube.com
+49 6222 – 386265
skillqube.com/support